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**IMPORTANT: BEFORE USING THE PRODUCT, PLEASE READ THESE INSTRUCTIONS CAREFULLY.**

### **GENERAL INSTRUCTIONS**

These instructions for use cannot replace the training, instruction, care, and knowledge of healthcare personnel. All relevant legal provisions, standards, regulations, and recommendations must be known to the healthcare staff.

### **DESCRIPTION**

The GHOTO thermal flask is a reusable, thermally insulating vessel made of 316L stainless steel with double walls and a vacuum interspace. It is designed to maintain the temperature of the saline solution poured inside. The silicone base ensures stability, the silicone disc placed at the bottom reduces the risk of damaging optics, and the silicone cap guarantees closure, minimizes heat loss, and is also used as support for the optics.

### **INTENDED USE**

For all surgical procedures where warm saline solution is required, particularly in laparoscopic and thoracoscopic surgeries, to warm or clean optics.

### **ILLUSTRATIONS AND NOMENCLATURE (Fig.1)**

- A. Anti-tilt silicone base
- B. 316L stainless steel thermal flask
- C. Silicone optical protection disc
- D. Lid with anchored silicone cap
- E. Hole for inserting the optic

### **HOW TO USE**

Place the thermal flask (B) on the anti-tilt silicone base (A) with slight pressure. Place the silicone optical protection disc (C) at the bottom of the flask. Pour 250 ml of warm saline solution into the flask. Close the flask with the lid (D) with slight pressure with cap open. Insert the optics through the hole (E) after removing the cap and reseal it after each cleaning.

### **MALFUNCTIONING**

In case of malfunction, immediately contact the manufacturer at [info@alker.it](mailto:info@alker.it), specifying the model in use and the cause of malfunction.

### **MAINTENANCE**

⚠ **WARNING:** Check, thoroughly inspect, clean, and sterilize the product before the first use, as the product is supplied non-sterile.

### **DEVICE FEATURES**

The GHOTO Medical Device is supplied non-sterile. It is reusable and has been tested for 50 reprocessing cycles.

### **LATEX**

The device is latex-free.

### **SPARE PARTS**

If necessary, use only original spare parts.

### **MANUAL WASHING**

**Decontamination:** After use, immerse and disassemble the devices in a solution containing a disinfectant chemical agent to decontaminate them. Carefully follow the activation times, immersion times, and concentrations indicated on the disinfectant label. After this period, remove the device and rinse it under running water.

**Washing:** After the decontamination phase, immerse the disassembled device in a detergent solution (or detergent-disinfectant). Follow the activation times, immersion times, and concentrations indicated on the detergent label. Remove attached impurities using a soft cloth or brush.

**Rinsing:** Rinse the material under running water to remove detergent residues.

**Drying:** After rinsing, thoroughly dry the material using paper or fabric cloths that do not shed fibers. It is preferable to use compressed air guns to eliminate water residues.

### **AUTOMATIC WASHING**

Automatic washing can be performed with all parts disassembled using thermal disinfectors with standardized programs up to 95°C. Neutral pH, mild-alkaline, or alkaline detergents can be used. Follow the concentrations indicated on the detergent label.

### **STERILIZATION**

⚠ **CAUTION:** Sterilization does not replace cleaning.

⚠ **CAUTION:** Do not assemble the GHOTO during the sterilization process.

**Sterilization:** Only sterilize products that have been cleaned and disinfected. Sterilize in an autoclave (fractionated pre-vacuum process) or plasma gas sterilizer. Follow the EN ISO 17665 standard for autoclave sterilization. Observe national guidelines.

**Process:** Fractionated vacuum (dynamic evacuation), with at least 3 pre-vacuum phases and a pressure of at least 60 mbar.

**Sterilization temperature:** min 132°C, max 138°C.

**Sterilization time: (time spent at sterilization temperature)** min 3 minutes, max 18 minutes.

**Drying time:** minimum 10 minutes.

 **The conformity of the sterilization process is the responsibility of the user.**

## **PRECAUTIONS AND WARNINGS**

1. Place the GHOTO on a stable and perfectly horizontal surface.
2. Risk of burns. Avoid impacts during use to prevent hot water spills.
3. Risk of burns. Do not lift the flask by the lid.
4. Risk of burns. The flask contains very hot water during use. Accidental contact with the patient or operators may cause burns.
5. Place the silicone disc at the bottom of the flask to avoid damaging the optics.
6. Do not damage or puncture the flask, as its functionality would be compromised.
7. Use distilled water-based solutions for cleaning during use and maintenance.
8. Clean thoroughly before the first use.
9. Periodically inspect the device for signs of wear or misalignment of components caused by improper handling or accidental drops.
10. Do not assemble the device during sterilization.
11. The device should only be used by qualified and trained healthcare professionals.
12. The device is not intended, sold, or indicated for uses other than those specified.
13. The device must not be used for purposes other than its intended use.
14. Healthcare professionals should pay attention to all warnings, contraindications, and precautions included in the "Instructions for Use" in every package.
15. The medical device is supplied non-sterile.
16. Before using the device, check that the package is not damaged and that all components are intact.

## **STORAGE CONDITIONS**

**GHOTO** Medical Device has no special storage conditions and can be stored at temperatures between 0°C and 80°C.

## **DISPOSAL**

Comply with local and national laws and regulations.

## **WARRANTY**

This product is designed and manufactured according to the best market standards. The warranty covers only manufacturing defects of this product. This product should only be used by skilled surgeons familiar with the adopted surgical technique. The warranty does not cover direct or indirect liability for any accidents or losses caused by the use of this product. No warranty covers any improper use of the product.

## **REFERENCE STANDARDS**

This medical device is designed and manufactured according to Regulation (EU) 2017/745 and the UNI CE EN ISO 13485:2021 standards, and belongs to Class I.

## **EXPLANATION OF SYMBOLS USED**

	Catalogue number
	Batch code
	Manufacturer
	CE Marking
	Caution
	Not sterile
	Latex free
	Consult Instructions for Use
	Storage temperature range

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